

# **Test Definition: FMETX**

Methsuximide (Celontin) as Desmethylmethsuximide

**Reporting Title:** Methsuximide (Celontin) **Performing Location:** Medtox Laboratories, Inc.

#### **Specimen Requirements:**

Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), plasma gel tube is not acceptable. Spin down and send 2 mL sodium heparin plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), serum gel tube is not acceptable. Spin down and send 2 mL of serum refrigerated in a plastic vial.

# **Specimen Minimum Volume:**

0.25 mL

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	7 days	
	Frozen	180 days	
	Ambient	72 hours	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z1107	Methsuximide	Alphanumeric		3801-8

LOINC and CPT codes are provided by the performing laboratory.

## Supplemental Report:

No

#### **CPT Code Information:**

80339



# **Test Definition: FMETX**

Methsuximide (Celontin) as Desmethylmethsuximide

## **Reference Values:**

10.0 - 40.0 ug/mL

Methsuximide measured as desmethylmethsuximide.