

Test Definition: POSV

Post Vasectomy Check, Semen

Reporting Title: Post Vasectomy Check, Semen

Performing Location: Rochester

Shipping Instructions:

Specimen must arrive within 24 hours of collection. Send specimen Monday through Thursday only and not the day before a holiday. If holiday falls on a Saturday, holiday will be observed on the preceding Friday. Sunday holidays are observed on the following Monday. Specimen should be collected and packaged as close to shipping time as possible. Laboratory does not perform testing on weekends.

Necessary Information:

Include the following information with the specimen:

- -Semen volume
- -Number of days of sexual abstinence

Specimen Requirements:

Patient Preparation: Patient should have 2 to 7 days of sexual abstinence at the time of semen collection for accurate

results.

Container/Tube: Semen Analysis Kit (T178)

Specimen Volume: Total ejaculate

Specimen Minimum Volume:

See Specimen Required

Specimen Type	Temperature	Time	Special Container
Semen	Ambient		

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
ABSTN	Abstinence	Numeric	d	10587-4
	Also used by tests: FER			
CLST1	Collection Site	Alphanumeric		56816-2
	Also used by tests: FER			



Test Definition: POSV

Post Vasectomy Check, Semen

Result ID	Reporting Name	Туре	Unit	LOINC®
VL53	Semen Volume	Numeric	mL	3160-9
	Also used by tests: FER			
CENT	Centrifuged	Alphanumeric		8100-0
SPML1	Sperm/mL	Numeric	x10(6)	51623-7
MOTIL	Motility	Numeric	%	6800-7
GR3	Grade	Numeric		29990-9
CMMT4	Comment	Alphanumeric		48767-8

LOINC and CPT codes are provided by the performing laboratory.

Supp	lemental	Report:
------	----------	---------

No

CPT Code Information:

89321

Reference Values:

Zero sperm seen