

# **Reporting Title:** Filaria, B **Performing Location:** Rochester

#### **Specimen Requirements:**

Container/Tube: Light-blue top (3.2% sodium citrate)

Specimen Volume: 2.7 mL

Collection Instructions: Certain microfilariae have a nocturnal or diurnal periodicity, therefore the blood collection should be timed to correspond with the release of microfilariae in the peripheral circulation. For the agents of lymphatic filariasis (Wuchereria bancrofti and the Brugia species), blood should be collected between 10 p.m. and 2 a.m., whereas for detection of Loa loa, blood should be collected between 10 a.m. and 2 p.m.

#### **Specimen Minimum Volume:**

2 mL

### Forms:

If not ordering electronically, complete, print, and send a Microbiology Test Request (T244) with the specimen.

Specimen Type	Temperature	Time
Whole Blood Na Cit	Ambient (preferred)	72 hours
	Refrigerated	72 hours

## Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
FIL	Q00M0022	Specimen Source	Plain Text	No

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
FIL	Filaria, B	Alphanumeric		10662-5

LOINC and CPT codes are provided by the performing laboratory.

#### Supplemental Report:

No



## **CPT Code Information:**

87015 87210

# **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
FILB	Filaria Bill Only			No	No (Bill Only)

## **Reference Values:**

Negative If positive, organism is identified.